


Receiver:
Start Here

G7

AW-00081-902 Rev 001
MT-00081-902
Rev Date 10/2022

G7 Basics | G7 shows your glucose information on your display devices.

Sensor and applicator

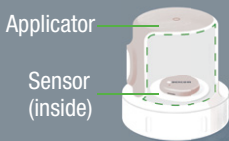

- During setup, we'll show you how to use the applicator to insert the built-in sensor under your skin
- The sensor sends a glucose reading to your receiver or other display device every 5 minutes
- The sensor lasts up to 10 days with a 12-hour grace period

Overpatch



After you insert the sensor, you must apply the overpatch to keep the sensor on your body.

Receiver setup

1. Insert sensor

- You must insert your sensor before pairing to the receiver.
- For instructions go to *Inserting Sensor* in the sensor box.

START HERE

1 G7 BASICS

G7 shows your glucose information on your display

2. Set up receiver

- Press the **Select** button for 3-5 seconds to turn on receiver. (Go to Receiver navigation.)
- Follow receiver instructions to set up the receiver. Insert sensor before pairing.
- After inserting and pairing, follow local guidelines for throwing out the applicator and recycling the Dexcom packaging.

3. Sensor warmup

Sensor warmup timer tells you when you'll start getting readings and alerts.



4. Your sensor session

Go to *Using Your Receiver* in the receiver box and the online *User Guide* at dexcom.com/guides to find out more.

The receiver screen tells you which button to use.

- To scroll faster hold down the **Scroll** button
- To move to next field use the **Select** button



Use up to 3 display devices

Get your glucose information on your phone, Apple watch, and the Dexcom receiver. You can set up the app, the receiver, or both, in either order.

Review Safety Information in product instructions. All graphics are representational. Your product may look different.

Dexcom

© 2022 Dexcom, Inc. All rights reserved.

Covered by patents [dexcom.com/patents](https://www.dexcom.com/patents).

Dexcom, Dexcom Share, Share, Dexcom Follow, and Dexcom Clarity are registered trademarks of Dexcom, Inc. in the United States and/or other countries. *Bluetooth* is a registered trademark owned by Bluetooth SIG. Apple is a trademark of Apple Inc., registered in the U.S. and other countries. Android is a trademark of Google LLC. All other marks are property of their respective owners.



Dexcom, Inc.
6340 Sequence Drive
San Diego, CA 92121 USA
1.888.738.3646
[dexcom.com](https://www.dexcom.com)

